


Short instructions for use

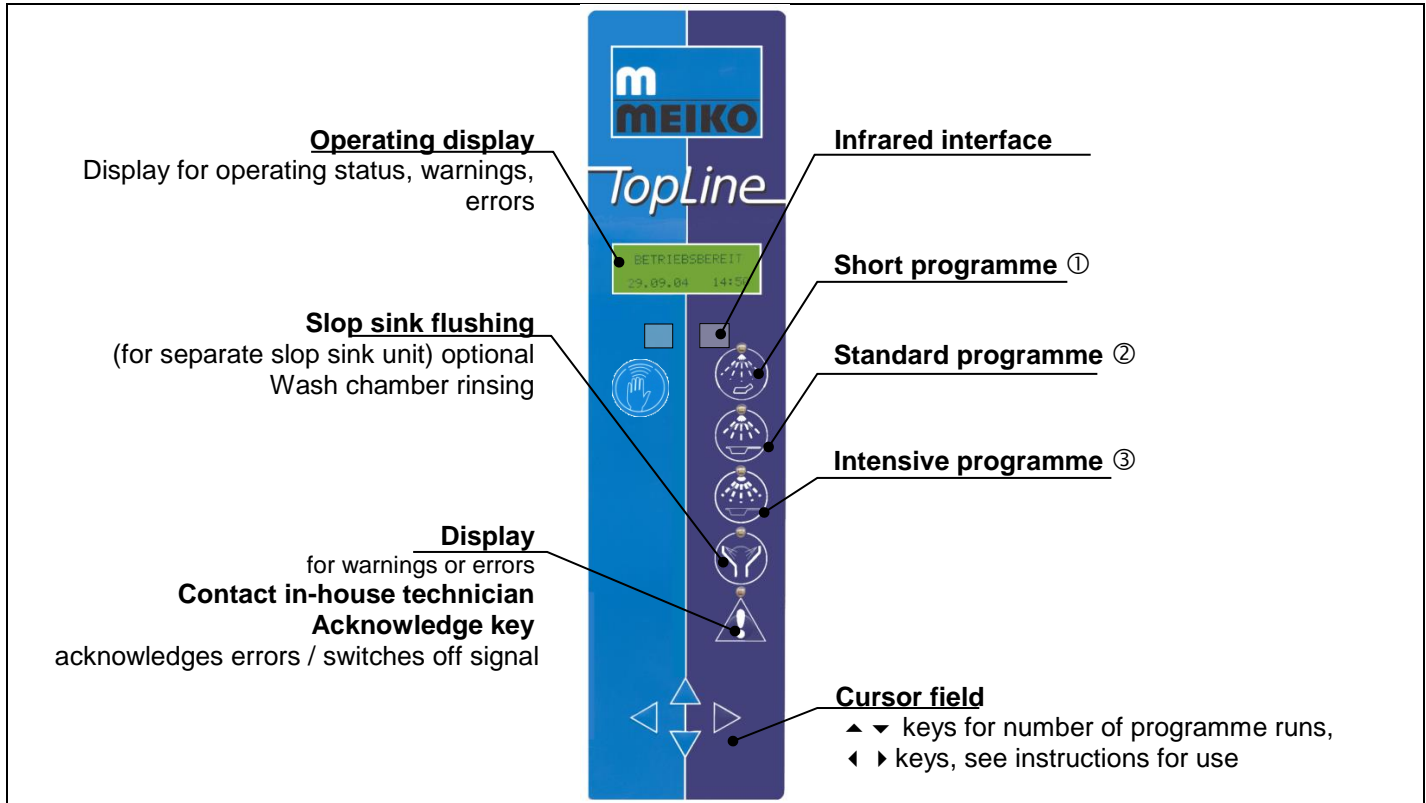

1. Open wash chamber door using the light sensor or foot switch.



2. Place the washware in the intended position in the utensil holder.



3. Select the cleaning programme corresponding to dirt. Press the programme key. The door closes and the programme starts with the preset A₀-value automatically.



4. ALTERNATIVE

Hold the programme key until the desired value A₀ is displayed centrally in []. Then immediately release the programme key. The door closes and the programme starts automatically.



Caution

Crushing hazard

Door movement can crush fingers, hands and arms.

- Do not reach into the machine when the door is in motion.
- Call in-house technician or MEIKO service technician to remove any door blockages.